

Quantisal™ Oral Fluid Collection Device for Oral Drug Tests



COLLECTION PROCESS:

STEP 1:

- Positively ID the donor.
- Explain the test to the donor.
- Inspect the collection device.
- Open up the packaging containing the collection pad & specimen vial.

STEP 2:

- Place the vial in front of the donor.
- Have the donor grasp the handle and remove the collection device from the sleeve.

STEP 3:

- Instruct the donor to place the pad under the tongue.
- Remove the device when the handle tip begins to turn blue.



STEP 4:

- Carefully open the vial in the upright position.
- Give the donor the vial and have them push the collection pad into the vial.
- Have the donor place the cap on the vial until the top snaps.



STEP 5:

- Have the donor place the tamper-evident seal across the top of the vial.
- The donor should date and initial the vial.
- Ensure that the first page of the Chain-of-Custody is with the sample vial.